



Kangaroo™ Connect Enteral Feeding Pump Quick Start Directions for Use

Pump Overview

1. SOFT KEYS


Press a key to select the option that appears next to it on the screen.

2. POWER BUTTON

Press once to power on.
Press and hold to power off.

3. POWER SOURCE

A green light shines when the pump is connected to AC power.

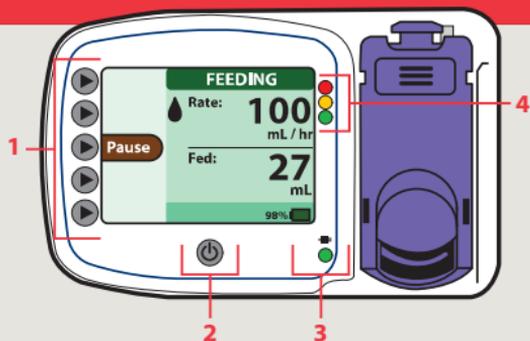


4. PUMP STATUS


- = Warning Alarm
- = Notice or Caution Alarm
- = Standby or Feeding

Pump Set Loading and Priming Procedure


1. Press the **POWER** button.

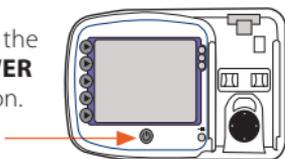

3. Select **PRIME**.

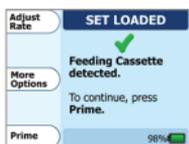


2. **LOAD SET**
(insert from the bottom and snap into place).

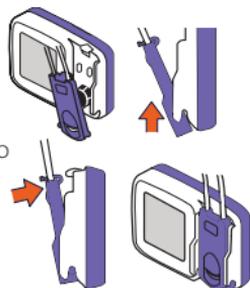


Prime the Pump


1. Select **PRIMING** mode.

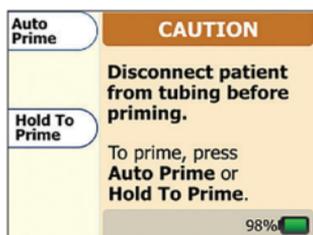

2. If you selected **AUTO PRIME**:
Wait for pump to finish priming.

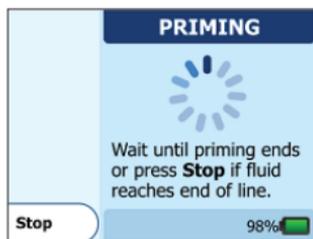


3. If you selected **HOLD TO PRIME**:
Press and hold **PRIME FEED** until fluid reaches end of tubing.

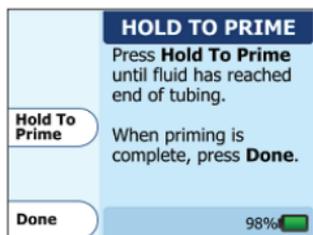


Ensure the Correct Fluid Level

Make sure the top of the fluid column is placed at 25.4cm above the top of the pump.

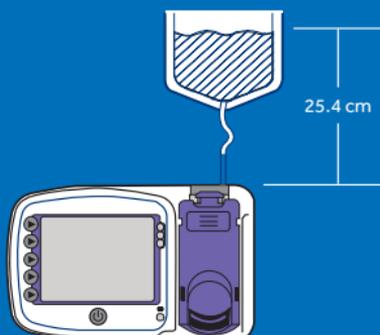


Start Feeding

1. Select **CLEAR SETTINGS**.

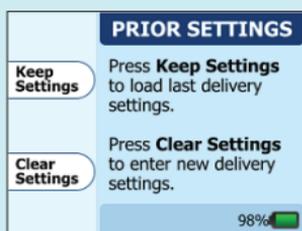

2. Use the Plus (+) and minus (-) buttons to enter the rate, then select **OK**.

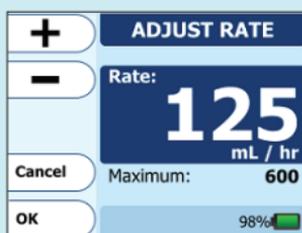

3. When ready to begin feeding, select **START**.



4. To suspend the feeding, select **PAUSE**.



5. To resume feeding, select **CONTINUE**.





Alarms and Troubleshooting


1. The **FEED COMPLETE** screen will appear if a Dose or Bolus feeding meets the specified amount. Press Done to clear the notification.




2. The **FEED INCOMPLETE** screen will appear if a Dose or Bolus feeding is interrupted prior to completion of the allocated amount. Press Resume Dose or Resume Boluses to continue feeding from the current pump state. Press Restart Bolus or Restart Dose if you are beginning a new, full feeding regimen.

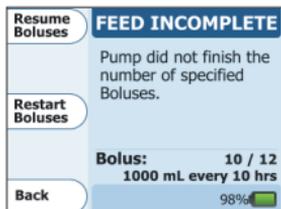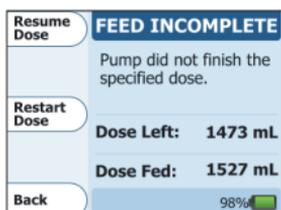


3. The **PUMP INACTIVE** error screen will appear if the pump has been without input for more than 10 minutes. Press Continue to return to the previous screen.

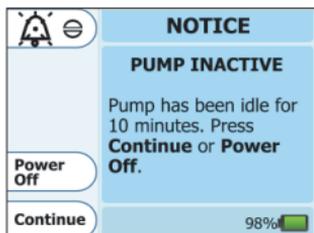


4. The **LOW BATTERY** screen appears and the alarm beeps continuously when the battery needs to be recharged. There is approx. 30 minutes of battery life remaining when this screen appears. Plug the power adaptor into a wall outlet to begin charging.

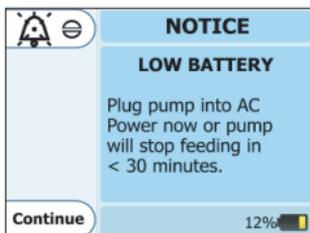

5. The **FEED BAG EMPTY** screen appears when the enteral formula is no longer being delivered because the bag is empty. Check the bag to see if it is empty and re-fill the bag as required.

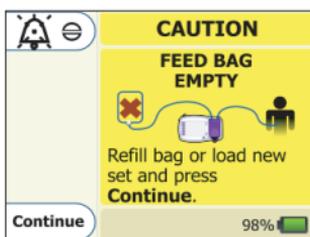


6. The **ROTOR STUCK** error screen appears when the pump detects an unusual amount of resistance to the rotation of the rotor. Clean the rotor and rotor shaft.

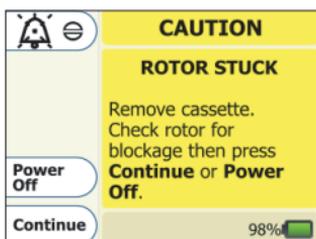


Alarms and Troubleshooting cont.


7. The **PATIENT TUBE BLOCKED** error screen appears when the feeding solution is no longer being delivered because of a clog between the pump and the patient. If the error cannot be resolved, remove the cassette from the pump. Check the line to find and clear the blockage.

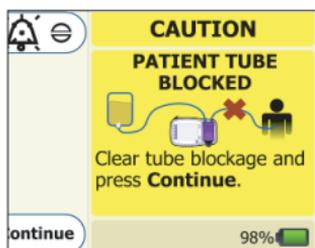


8. The **SUPPLY TUBE BLOCKED** screen will appear if there is a blockage, obstruction or kinked tubing between the feeding bag and the pump. Check the tubing between the bag and the pump to see if the cause of the blockage can be located and cleared.

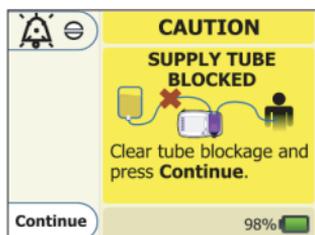


9. The **CASSETTE DISLODGED** screen will appear if the magnet in the cassette is not properly loaded in the pump set loading area. Reload the cassette to ensure the correct positioning of the cassette on the pump.

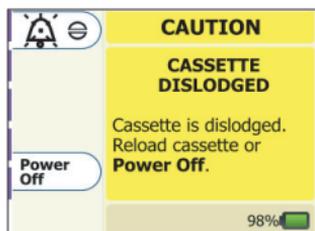

10. The **CASSETTE ERROR** screen appears during Feeding or Prining when the pump has detected an unusual operating condition with the rotor. Check that the pump set is not damaged and re-load the cassette. Press **CONTINUE** to Start.

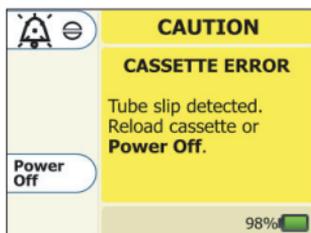

11. If the Low Battery screen has been displayed for some time without a response, the pump will switch to a **DEAD BATTERY ERROR** alarm status. Plug the pump into a wall outlet.

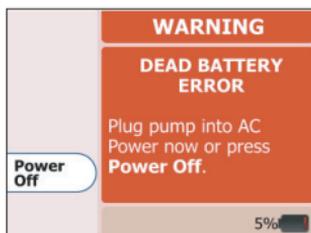

cardinalhealth.com.au

Cardinal Health Australia
Level 2, 5 Eden Park Drive, North Ryde NSW 2113

© 2019 Cardinal Health. All Rights Reserved. CARDINAL HEALTH, the Cardinal Health LOGO and ESSENTIAL TO CARE are trademarks of Cardinal Health and may be registered in the US and/or in other countries. All other marks are the property of their respective owners. 1434-03-19

